

FITTING GUIDELINES

Focus DAILIES Toric with AquaComfort


4 NEW AXES

Confidently fit Focus DAILIES Toric with AquaComfort as your contact lens of choice for more astigmatic customers.* It's the daily disposable toric contact lens to deliver sustained, blink-activated moisture release for all day comfort.

- NEW** additional axes—20°, 70°, 110°, 160°
- NEW** scribe marks for accurate fitting
- NEW** inversion mark to aid contact lens insertion
- NEW** 90-pack offers greater value




More parameters for more customers*

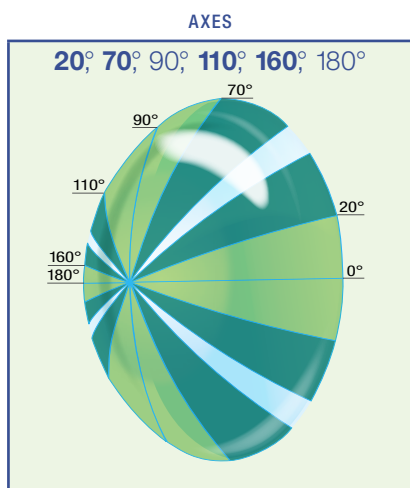

Customer coverage based upon sphere power, cylinder and axis

78%

Convenient fit with one base curve and new scribe marks

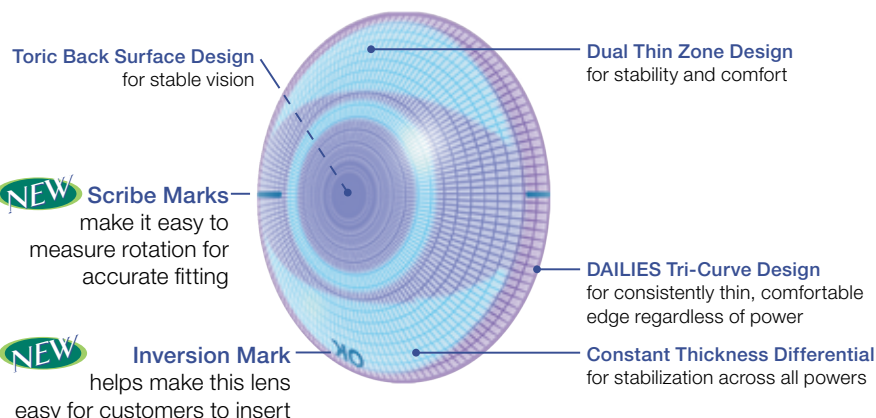

Focus DAILIES Toric with AquaComfort Product Details

Material:	Nelfilcon A; 69% water	Sphere powers (D):	+4,00D to -6,00D (0,25D steps) -6,50D to -8,00D (0,50D steps)	THE ONLY DAILY DISPOSABLE TORIC LENS WITH PLUS POWERS
Comfort enhancer:	AquaComfort	Cylinder Powers (D):	-0,75D; -1,50D	
Handling tint:	Light blue Visitint	Axes:	NEW 20°; 70°; 90°; 110°; 160°; 180°	
Diameter:	14,2 mm	Scribe marks:	NEW Scribe marks at 3 and 9 o'clock	
Base curve:	8,6 mm	Inversion mark:	NEW OK inversion mark for easy insertion	
Dk/t:	26	Design:	Dual thin zone, Toric back surface design	
Center thickness:	0,10 mm @ -3,00D (varies with power)	Wearing schedule:	Daily wear	


THE ONLY DAILY DISPOSABLE TORIC LENS WITH AQUACOMFORT




The secret is in the blink

Follow these easy steps to fit successfully

1. Select Initial Contact Lens

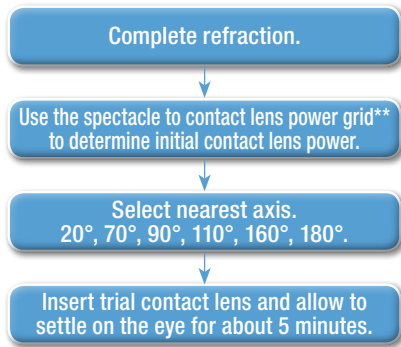

2. Evaluate Contact Lens Fit

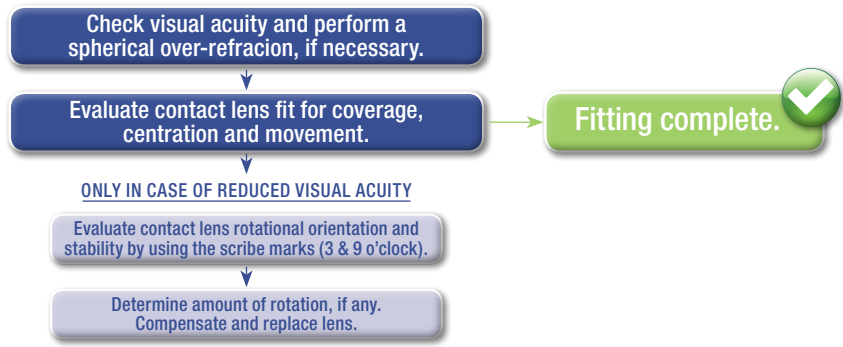

Spectacle Cylinder Powers[†]

	-0,75		-1,00		-1,25		-1,50		-1,75		-2,00		-2,25	
	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl	sph	cyl
-9,25	-8,00	-0,75	-8,00	-0,75	-8,00	-0,75	-8,00	-1,50	-8,00	-1,50	-8,00	-1,50	-8,00	-1,50
-9,00	-8,00	-0,75	-8,00	-0,75	-8,00	-0,75	-8,00	-1,50	-8,00	-1,50	-8,00	-1,50	-8,00	-1,50
-8,75	-8,00	-0,75	-8,00	-0,75	-8,00	-0,75	-8,00	-1,50	-8,00	-1,50	-8,00	-1,50	-8,00	-1,50
-8,50	-7,50	-0,75	-7,50	-0,75	-8,00	-0,75	-7,50	-1,50	-7,50	-1,50	-8,00	-1,50	-8,00	-1,50
-8,25	-7,50	-0,75	-7,50	-0,75	-7,50	-0,75	-7,50	-1,50	-7,50	-1,50	-7,50	-1,50	-7,50	-1,50
-8,00	-7,00	-0,75	-7,50	-0,75	-7,50	-0,75	-7,00	-1,50	-7,50	-1,50	-7,50	-1,50	-7,50	-1,50
-7,75	-7,00	-0,75	-7,00	-0,75	-7,00	-0,75	-7,00	-1,50	-7,00	-1,50	-7,00	-1,50	-7,50	-1,50
-7,50	-7,00	-0,75	-7,00	-0,75	-7,00	-0,75	-6,50	-1,50	-7,00	-1,50	-7,00	-1,50	-7,00	-1,50
-7,25	-6,50	-0,75	-6,50	-0,75	-7,00	-0,75	-6,50	-1,50	-6,50	-1,50	-7,00	-1,50	-7,00	-1,50
-7,00	-6,50	-0,75	-6,50	-0,75	-6,50	-0,75	-6,50	-1,50	-6,50	-1,50	-6,50	-1,50	-6,50	-1,50
-6,75	-6,00	-0,75	-6,50	-0,75	-6,50	-0,75	-6,00	-1,50	-6,00	-1,50	-6,50	-1,50	-6,50	-1,50
-6,50	-6,00	-0,75	-6,00	-0,75	-6,00	-0,75	-6,00	-1,50	-6,00	-1,50	-6,25	-1,50	-6,25	-1,50
-6,25	-5,75	-0,75	-5,75	-0,75	-6,00	-0,75	-5,75	-1,50	-5,75	-1,50	-6,00	-1,50	-6,00	-1,50
-6,00	-5,50	-0,75	-5,75	-0,75	-5,75	-0,75	-5,50	-1,50	-5,50	-1,50	-5,75	-1,50	-5,75	-1,50
-5,75	-5,25	-0,75	-5,50	-0,75	-5,50	-0,75	-5,25	-1,50	-5,50	-1,50	-5,50	-1,50	-5,50	-1,50
-5,50	-5,00	-0,75	-5,25	-0,75	-5,25	-0,75	-5,00	-1,50	-5,25	-1,50	-5,25	-1,50	-5,50	-1,50
-5,25	-4,75	-0,75	-5,00	-0,75	-5,00	-0,75	-4,75	-1,50	-5,00	-1,50	-5,00	-1,50	-5,25	-1,50
-5,00	-4,75	-0,75	-4,75	-0,75	-4,75	-0,75	-4,50	-1,50	-4,75	-1,50	-4,75	-1,50	-5,00	-1,50
-4,75	-4,50	-0,75	-4,50	-0,75	-4,50	-0,75	-4,50	-1,50	-4,50	-1,50	-4,75	-1,50	-4,75	-1,50
-4,50	-4,25	-0,75	-4,25	-0,75	-4,25	-0,75	-4,25	-1,50	-4,25	-1,50	-4,50	-1,50	-4,50	-1,50
-4,25	-4,00	-0,75	-4,00	-0,75	-4,00	-0,75	-4,00	-1,50	-4,00	-1,50	-4,25	-1,50	-4,25	-1,50
-4,00	-3,75	-0,75	-4,00	-0,75	-4,00	-0,75 [‡]	-3,75	-1,50	-3,75	-1,50	-4,00	-1,50	-4,00	-1,50
-3,75	-3,50	-0,75	-3,75	-0,75	-3,75	-0,75 [‡]	-3,50	-1,50	-3,75	-1,50	-3,75	-1,50	-3,75	-1,50
-3,50	-3,25	-0,75	-3,50	-0,75	-3,50	-0,75 [‡]	-3,25	-1,50	-3,50	-1,50	-3,50	-1,50	-3,75	-1,50
-3,25	-3,00	-0,75	-3,25	-0,75	-3,25	-0,75 [‡]	-3,00	-1,50	-3,25	-1,50	-3,25	-1,50	-3,50	-1,50
-3,00	-2,75	-0,75	-3,00	-0,75	-3,00	-0,75 [‡]	-2,75	-1,50	-3,00	-1,50	-3,00	-1,50	-3,25	-1,50
-2,75	-2,75	-0,75	-2,75	-0,75	-2,75	-0,75 [‡]	-2,50	-1,50	-2,75	-1,50	-2,75	-1,50	-3,00	-1,50
-2,50	-2,50	-0,75	-2,50	-0,75	-2,75	-0,75 [‡]	-2,50	-1,50	-2,50	-1,50	-2,50	-1,50	-2,75	-1,50
-2,25	-2,25	-0,75	-2,25	-0,75	-2,50	-0,75 [‡]	-2,25	-1,50	-2,25	-1,50	-2,50	-1,50	-2,50	-1,50
-2,00	-2,00	-0,75	-2,00	-0,75	-2,25	-0,75 [‡]	-2,00	-1,50	-2,00	-1,50	-2,25	-1,50	-2,25	-1,50
-1,75	-1,75	-0,75	-1,75	-0,75	-2,00	-0,75 [‡]	-1,75	-1,50	-1,75	-1,50	-2,00	-1,50	-2,00	-1,50
-1,50	-1,50	-0,75	-1,50	-0,75	-1,75	-0,75 [‡]	-1,50	-1,50	-1,50	-1,50	-1,75	-1,50	-1,75	-1,50
-1,25	-1,25	-0,75	-1,25	-0,75	-1,50	-0,75 [‡]	-1,25	-1,50	-1,25	-1,50	-1,50	-1,50	-1,50	-1,50
-1,00	-1,00	-0,75	-1,00	-0,75	-1,25	-0,75 [‡]	-1,00	-1,50	-1,00	-1,50	-1,25	-1,50	-1,25	-1,50
-0,75	-0,75	-0,75	-0,75	-0,75	-1,00	-0,75 [‡]	-0,75	-1,50	-0,75	-1,50	-1,00	-1,50	-1,00	-1,50
-0,50	-0,50	-0,75	-0,50	-0,75	-0,75	-0,75 [‡]	-0,50	-1,50	-0,50	-1,50	-0,75	-1,50	-0,75	-1,50
-0,25	-0,25	-0,75	-0,25	-0,75	-0,50	-0,75 [‡]	-0,25	-1,50	-0,25	-1,50	-0,50	-1,50	-0,50	-1,50
0,00	0,00	-0,75	0,00	-0,75	-0,25	-0,75 [‡]	0,00	-1,50	0,00	-1,50	-0,25	-1,50	-0,25	-1,50
+0,25	+0,25	-0,75	+0,25	-0,75	0,00	-0,75 [‡]	+0,25	-1,50	+0,25	-1,50	0,00	-1,50	0,00	-1,50
+0,50	+0,50	-0,75	+0,50	-0,75	+0,75	-1,50	+0,50	-1,50	+0,50	-1,50	+0,25	-1,50	+0,25	-1,50
+0,75	+0,75	-0,75	+0,75	-0,75	+1,00	-1,50	+0,75	-1,50	+0,75	-1,50	+0,50	-1,50		
+1,00	+1,00	-0,75	+1,00	-0,75	+1,25	-1,50	+1,00	-1,50	+1,00	-1,50	+0,75	-1,50		
+1,25	+1,25	-0,75	+1,25	-0,75	+1,50	-1,50	+1,25	-1,50	+1,25	-1,50	+1,00	-1,50		
+1,50	+1,50	-0,75	+1,50	-0,75	+1,75	-1,50	+1,50	-1,50	+1,50	-1,50	+1,25	-1,50		
+1,75	+1,75	-0,75	+1,75	-0,75	+2,00	-1,50	+1,75	-1,50	+1,75	-1,50	+1,50	-1,50		
+2,00	+2,00	-0,75	+2,00	-0,75	+2,25	-1,50	+2,00	-1,50	+2,00	-1,50	+1,75	-1,50		
+2,25	+2,25	-0,75	+2,25	-0,75	+2,50	-1,50	+2,25	-1,50	+2,25	-1,50	+2,00	-1,50		
+2,50	+2,50	-0,75	+2,50	-0,75	+2,75	-1,50	+2,50	-1,50	+2,50	-1,50	+2,25	-1,50		
+2,75	+2,75	-0,75	+2,75	-0,75	+3,00	-1,50	+2,75	-1,50	+2,75	-1,50	+2,50	-1,50		
+3,00	+3,00	-0,75	+3,00	-0,75	+3,25	-1,50	+3,00	-1,50	+3,00	-1,50	+2,75	-1,50		
+3,25	+3,25	-0,75	+3,25	-0,75	+3,50	-1,50	+3,25	-1,50	+3,25	-1,50	+3,00	-1,50		
+3,50	+3,75	-0,75	+3,50	-0,75	+3,75	-1,50	+3,50	-1,50	+3,50	-1,50	+3,25	-1,50		
+3,75	+4,00	-0,75	+3,75	-0,75	+4,00	-1,50	+4,00	-1,50	+3,75	-1,50	+3,75	-1,50		
+4,00	+4,00	-0,75	+4,00	-0,75	+4,00	-1,50	+4,00	-1,50	+4,00	-1,50	+4,00	-1,50		

Trial Contact Lens Powers

Trial Contact Lens Powers

*Compared to previously available Focus DAILIES Toric parameter range, based on the prevalence of refractive errors and calculation of residual astigmatism.
 **Determination of which final cylinder axis to order will be made after compensation for contact lens orientation.
 †-1,50D cylinder optional

